HINDI

Calypso operating instructions and pumpset assembly.

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6. Insert the tube connector into the pump and you are ready to start expressing.

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Side 2– Operating the Calypso Breastpump

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7) Connect the power supply to the Breastpump.

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8) Switch on the Calypso pump by pressing the on/off switch – green LED light will come on – vacuum & cycle levels are at lowest intensity and can be modified individually.

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9) Press the up arrow to increase the vacuum (photo on the left) and cycle (photo on the right) levels and the down arrow to reduce the level(s). Each touch increases/decreases the levels by one step. When babies start to breast feed, they suckle with rapid sucking motions and low suction - this causes the milk to release by stimulating let down. This effect can be replicated by starting with a higher cycle level and selecting a weaker vacuum (stimulation mode). When using stimulation mode, choose the levels of vacuum and cycle which suit you.

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10) When let down occurs, reduce the cycles and strengthen the vacuum until you find the most comfortable level for you.

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Vary the cycles and vacuum levels to maintain the milk flow and feel comfortable, as this mimics the natural sucking rhythm of the baby.

Ardo medical Ltd is proud to declare its total commitment to the WHO CODE

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If you need any further help please refer to the Calypso user manual.

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