Polish

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Side 2– Operating the Calypso Breastpump

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7) Connect the power supply to the Breastpump.

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8) Switch on the Calypso pump by pressing the on/off switch – green LED light will come on – vacuum & cycle levels are at lowest intensity and can be modified individually.

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9) Press the up arrow to increase the vacuum (photo on the left) and cycle (photo on the right) levels and the down arrow to reduce the level(s). Each touch increases/decreases the levels by one step. When babies start to breast feed, they suckle with rapid sucking motions and low suction - this causes the milk to release by stimulating let down. This effect can be replicated by starting with a higher cycle level and selecting a weaker vacuum (stimulation mode). When using stimulation mode, choose the levels of vacuum and cycle which suit you.

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10) When let down occurs, reduce the cycles and strengthen the vacuum until you find the most comfortable level for you.

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Vary the cycles and vacuum levels to maintain the milk flow and feel comfortable, as this mimics the natural sucking rhythm of the baby.

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Ardo medical Ltd is proud to declare its total commitment to the WHO CODE

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If you need any further help please refer to the Calypso user manual.

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Firma Ardo Medical Ltd z dumą zobowiązuje się do przestrzegania Kodeksu Międzynarodowej Organizacji Zdrowia (WHO CODE).