

SUCTION TECHNOLOGY

Hydrophobic Bacterial Filter for all Ardo suction pumps

- **Double Function**
The filter closes in case of overflow or foam building and protects the inside of the pump. The efficient bacterial filter has a retention of 99.99946 % given a particle size of 3.3 μm .
- **Exchange of Filter**
Change the filter in case of discoloration or overflow so that the suction efficiency of the pump is guaranteed. The filter must be changed at least once a month.
- **Foot Operated Vacuum Regulator**
Connecting the foot vacuum regulator between pump and bacteria filter also protects the foot vacuum regulator against overflow and foam.

SWISS MADE 



Ardo Bacterial Filter / 12.2017 / V01