

Obstetric Products

Reliable and cost effective



Urine Specimen Collection Bags

The Sterile Urine Specimen Bags are the first choice when it comes to convenience, protection and care for delicate skin. The bags allow collection of an aseptic specimen for urine tests, without contamination.

200 ml Paediatric

Product Code:
Microporous 7531
Standard 7511
(Packs of 100)



100 ml Newborn

Product Code:
Microporous 7535
Standard 7515
(Packs of 100)




10 ml Preemie

Product Code:
Microporous 7540
(Packs of 100)



Microporous Adhesive

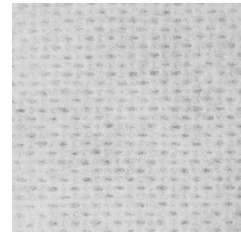


Standard Adhesive



What is the product?	Urine Specimen Collection Bags
When is it used?	During the Newborn and Paediatric period
What is the purpose?	To collect a urine specimen without contamination
How is it used?	Clean and thoroughly dry the skin. Remove release liner from lower half of adhesive patch. Apply the exposed portion of the adhesive with your fingers. Remove release liner from upper portion. Working upward and outward, smooth the adhesive against the skin, pressing firmly to avoid any wrinkles. Remove by gently peeling the adhesive from the infants skin. Discard appropriately.
What are the key features/benefits	<ul style="list-style-type: none"> • Different styles and sizes to meet a variety of clinical applications • The “Butterfly” adhesive patch minimises faecal contamination • Unisex design thus reducing stock holding for hospitals • Double Chamber construction with no flow back valve keeps urine away from the skin to help prevent irritation

Amniotic Membrane Perforator

The Amniotic Membrane Perforator provides a safe and comfortable method for rupturing the amniotic membrane.

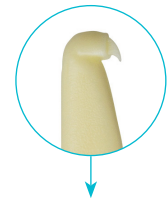

What is the product?	Amniotic Membrane Perforator Product Code: 9601 (Packs of 100)
When is it used?	During Labour
What is the purpose?	To perforate the amniotic sac to induce or hasten the delivery
How is it used?	With the protected "hook" pointing upward, slide the device into the cervix using the middle and index fingers of the examining hand as a guide. When the uterus contracts, press the rounded end of the perforator device against the membrane. Then gently draw the instrument out, snagging the membrane causing it to rupture. Slowly withdraw the device from the cervix cavity, using the protective groove formed by the fingers of the examining hand as a guide. Discard device.
What are the key features/benefits	<ul style="list-style-type: none"> • Moulded, non-slip gripping surface for easy handling • Rounded tip and protected point designed to rupture the amniotic membrane without injury to the mum or foetus • Disposable, pre-sterilised and individually packed for convenience

Umbilical Cord Clamp

This precision Umbilical Cord Clamp is hygienic, safe and reliable to use. After the birth, the umbilical cord can be clamped to prevent blood loss in the newborn baby.



What is the product?	Umbilical Cord Clamp Product Code: Sterile 65.00.51 Non Sterile 65.00.52 (Packs of 100)
What are the different types?	Available in Sterile and Non-Sterile form
When is it used?	Immediately after delivery
What does it do?	Clamps the umbilical cord prior to cutting the cord to stop blood loss. The umbilical cord has two veins and one artery: high risk of loss of blood without effective clamping.
What are the key features/benefits	<ul style="list-style-type: none"> • Safe, disposable, easy to apply • No rough edges to snag newborn clothing change or bed linen • Dual gripping services prevents slipping during application • Maintains tight seal and applies constant pressure on any size cord as the cord shrinks • Hinge guard confines cord between serrated jaws to eliminate movement.

Umbilical Cord Clamp Clipper

This specially designed Umbilical Cord Clamp Clipper safely removes the cord clamp from the baby by severing the hinge.



What is the product?	Umbilical Cord Clamp Clipper Product code: 9441 (Pack of 6)
When is it used?	After delivery
What is the purpose?	To safely remove the cord clamp from the baby
How is it used?	Cut cord clamp and safely remove from the umbilical cord
What are the key features/benefits?	<ul style="list-style-type: none">• Single use item to prevent the risk of cross contamination• Concealed cutting surface eliminates risk of harming baby• Safe and easy to use