CE
0123

 **ARDO**
Your Professional Way to Care

de
en
fr
it
nl
es
pt
tr
da
sv
no
fi
pl
gr

<u>Languages:</u>	<u>Page:</u>
Gebrauchsanweisung OneMum PumpSet 72h	2-3
Instructions for use OneMum PumpSet 72h	4-5
Mode d'emploi OneMum PumpSet 72h	6-7
Istruzioni per l'uso OneMum PumpSet 72h	8-9
Gebruiksaanwijzing OneMum PumpSet 72h	10-11
Indicaciones OneMum PumpSet 72h	12-13
Indicações OneMum PumpSet 72h	14-15
Kullanım kılavuzu OneMum PumpSet 72h	16-17
Brugsanvisning OneMum PumpSet 72h	18-19
Bruksanvisning OneMum PumpSet 72h	20-21
Bruksanvisning OneMum PumpSet 72h	22-23
Käyttöohjeet OneMum PumpSet 72h	24-25
Instrukcja OneMum PumpSet 72h	26-27
Οδηγίες χρήσης OneMum PumpSet 72h	28-29

OneMum PumpSet 72h

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OneMum PumpSet 72h

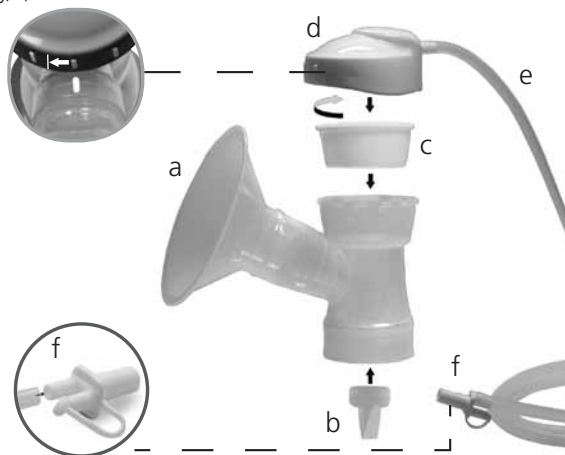
Sterile packed – designed to be used by one mother only, for up to 72 hours.

The OneMum PumpSet 72h can be used 7 to 8 times per day during and up to the 72 hour limit. The accessory, OneMum PumpSet 72h, is an applied part of the ARDO breastpump. It is intended for the expression of breast milk in the event of breastfeeding difficulties, illness or when mother and baby are separated.

The OneMum PumpSet 72h is sterile unless the packaging is damaged, or has been opened. These instructions apply to the accessory: "OneMum PumpSet 72h", with Breast Shell sizes: D26, D31 and D36.

Package Contents and Assembly:

a) Breast Shell, b) Lip valve, c) Membrane pot, d) Adapter tube cover, e) Tubing, f) Tube connector

Use:

- Always wash hands thoroughly with soap and water before handling.
- To express, sit comfortably and relax.
- Centre the nipple so that when expressing, it moves freely in the breast shell.
- Switch the breast pump on.

Cleaning after every use:

Clean the OneMum PumpSet 72h as follows:

1. Disassemble parts a) to c) before cleaning.
2. Rinse the parts with cold tap water to free or loosen the protein in the breast milk.
3. Wash the parts with warm soapy water. It is not necessary to use a brush to clean the inner parts.
4. Make sure that the soap residue has been thoroughly rinsed off.
5. Allow the OneMum PumpSet 72h to air dry.

Keep the OneMum PumpSet 72h in a clean bag – preferably in an ARDO AccessoryBag.

Parts d) to f) need only be cleaned when required by regular internal hospital procedures. If needed, clean with a disinfectant wipe. The inside of the tubing must stay dry.

Product life: 72 hours. However, the unit should be checked for damage each time it is assembled.

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Ardo medical AG