PUMPSETS AND ACCESSORIES

Instructions for use | Gebrauchsanweisung | Mode d'emploi | Istruzioni per l'uso | Instrucciones de uso | Indicações de utilização | Gebruiksaanwijzing | Bruksanvisning | Käyttöohje | Bruksanvisning | Brugsanvisning | Инструкция по эксплуатации | Instrukcja obsługi | Οδηγίες χρήσης | Kullanım Kılavuzu | دليل الاستعمال | 사용 설명서 | 使用说明

Languages**Page**

en	Instructions for use	Ardo Pumpsets and accessories	3–6
de	Gebrauchsanweisung	Ardo Pumpsets und Zubehör	7–10
fr	Mode d'emploi	des Pumpsets Ardo et accessoires	11–14
it	Istruzioni per l'uso	Pumpsets e accessori Ardo	15–18
es	Instrucciones de uso	Pumpsets Ardo y accesorios	19–22
pt	Indicações de utilização	Pumpsets e acessórios Ardo	23–26
nl	Gebruiksaanwijzing	Ardo Pumpsets en accessoires	27–30
sv	Bruksanvisning	Ardo Pumpset och tillbehör	31–34
fi	Käyttöohje	Ardo Pumpsets ja lisävarusteet	35–38
no	Bruksanvisning	Ardo Pumpsets og tilbehør	39–42
da	Brugsanvisning	Ardo Pumpsets og tilbehør	43–46
ru	Инструкция по эксплуатации	Pumpsets Ardo и его составляющих частей	47–50
pl	Instrukcja obsługi	Ardo Pumpsets i akcesoriów	51–54
el	Οδηγίες χρήσης	Ardo Pumpsets και αξεσουάρ	55–58
tr	Kullanım Kılavuzu	Ardo Pumpsets ve Aksesuarlar	59–62
ar	دليل الاستعمال	والمحقات التكميلية Ardo طقم المضخة	63–66
ko	사용 설명서	아르도 유축기 세트 및 부속품	67–70
zh	使用说明	Ardo 水泵机组及附件	71–74

1. Package contents

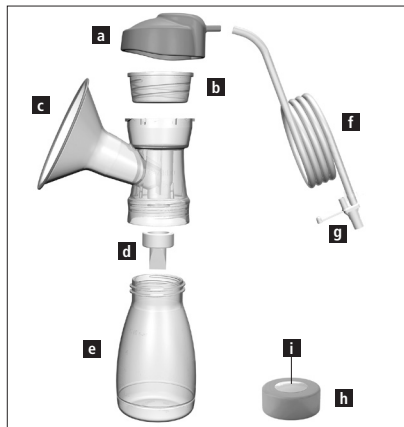
Pumpset:

- a.) Adapter tube cover
- b.) Flexible membrane pot
- c.) 26 mm breast shell
- d.) 1 lip valve plus 1 spare valve
- e.) 150 ml bottle
- f.) Tube
- g.) Tube connector
- h.) Cap
- i.) Cap liner

Double Pumpset: Two each of a to i


Clinic Pumpset: From a to i

Pumpset Kit: a, b, f and g

2. Supplementary accessories

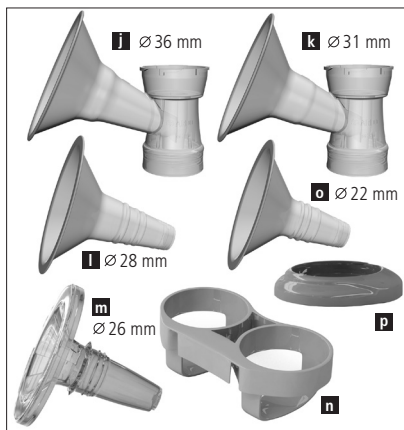The two Ardo breast shells j.) and k.) and the breast shell inserts l.) + o.) allow you to select your size individually. The Ardo Optiflow massage insert m.) guarantees an even gentler pumping experience and its massaging action stimulates the flow of milk (can only be used with the 31 mm breast shell). The bottle holder n.) is suitable for use with the Calypso Double Plus breastpump and the bottle stand p.) for Calypso.

3. Indications

The Ardo Pumpset should be used in combination with Ardo breastpumps and is intended for the expression, collection and storage of breast milk. The pumpset can be used as a single or double pumpset.

4. Contraindications

None known.

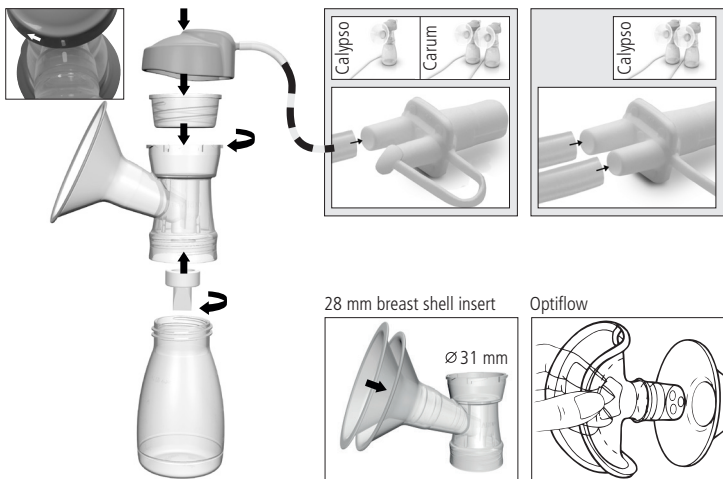
Safety regulations: Only use after cleaning or sterilisation. After cleaning check for any damage.

5. Assembling the pumpset

Assemble as illustrated.

Put on the adapter tube cover so that both marks coincide and then rotate to the right.

Important: Boil before first use (see section on "Cleaning at home") and check whether the flexible membrane pot is intact with the smooth side facing outwards

6. Cleaning at home

After each use, wash the pumpset with washing-up liquid and rinse well. Once a day, disassemble and boil the components for 10 minutes, or put in the dishwasher, microwave (in a microwave bag) or steam steriliser, then dry and cover with a clean cloth. Do not clean the lip valve with a brush.

The bottle holder should not be boiled.

7. Cleaning in hospitals

Disassemble and sterilise at 121°C (250°F) for 20 minutes / 134°C (273°F) for 5 minutes.

The bottle holder should not be sterilised.

The adapter tube cover, tube and tube connector may be boiled/sterilised but they do not have to be. It is sufficient to wipe them with a cloth when needed. It is not necessary to rinse the tube. However, it must always be completely dry before use.

8. Instructions for use

8.1 Expressing milk with one pumpset

- Wash your hands thoroughly.
- Attach the pumpset to the Ardo breastpump.
- Sit down comfortably and relax.
- Centre the nipple so that, when pumping, it moves freely in the breast shell (breast shell insert).
- Switch on the breastpump.

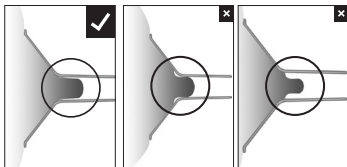
8.2 Expressing milk with a double pumpset

- Connect the second pumpset to the Ardo breastpump.
- Using the Ardo Calypso breastpump: Connect both pumpsets at the same time (see illustration). Switch on the pump and place one hand on each pumpset.
- Using the Ardo Carum breastpump: Place a pumpset on your breast, switch on the pump and position the second pumpset on your other breast. Alternatively, you can also position both pumpsets at the same time.

9. Choice of breast shell and Optiflow massage insert

If your nipple rubs uncomfortably on the inner side of the funnel, use a larger breast shell (breast shell insert). If the areola is also sucked into the funnel, use a smaller breast shell. The Optiflow can only be used with the 31 mm breast shell.

10. Storage of breast milk

Recommended storage time at room temperature: 3–4 hours at 16–29°C (60–85°F); in the refrigerator: 3 days at ≤4°C (39°F); in freezing compartment with door: 6 months to 1 year at <-17°C (0°F)

11. Possible problems and troubleshooting

Weak or no vacuum:

- Ensure that the pumpset has been correctly assembled and is not damaged.
- Check whether the lip valve is cracked or stuck together.
- Check whether the flexible membrane pot is intact and the smooth side is facing outwards.
- Make sure that the adapter tube cover seals well.

Pain when pumping:

- Ensure that the nipple is in the middle of the breast shell.
- Check that the breast shell size fits your nipples.
- Check that the vacuum intensity is not set too high.
- Seek professional help if problems persist.

12. General information

The accessory pumpset is an applied part of Ardo breastpumps. Lifetime of the pumpset: 400 hours or sterilised a maximum of 50 times.

M	Materials: Polypropylene (PP) / silicone
	Temperature: Operation +5°C (41°F) to +40°C (104°F) / Transport -20°C (-4 F°) to +50°C (122°F)
	Disposal: Can be safely disposed of in compliance with local regulations.
	Packaging: Caution, fragile

	Relative humidity: 20–95%
	Air pressure: 700–1060 hPa
	Keep packaging away from heat
	Keep packaging away from moisture

Preferably store Optiflow in a dark place (silicone).

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Ardo medical AG