

Tulips Nipple Shields

- Assists with sucking or latching on problems
- Delicate and soft silicone feels completely natural
- Maximum skin contact with unique tulip shape
- Ideal for unusual shaped nipples
- Easier for premature babies to suck
- Relieves sore, painful nipples
- Odourless and Latex free
- Soft shaft for the sensitive oral cavity of babies



Tulip nipple shields help assist latching-on problems and support pain-free breastfeeding, protecting and relieving severely stressed nipples.

The special tulip shape means that the baby retains maximum contact with the breast.

Tulips Nipple Shields - one pair packaged with storage box

Small (16mm)	63.00.323
Medium (20mm)	63.00.204
Large (24mm)	63.00.201



Tulips Nipple Shields - Packs of 10 - Sterile and Non-Sterile


	Pack of 10 singles Sterile packaging	Pack of 10 pairs Individually packaged
Small (16mm)	63.00.332H	63.00.23H
Medium (20mm)	63.00.284H	63.00.15H
Large (24mm)	63.00.285H	63.00.16H




Instructions

Application

1. Moisten the inside of the nipple shield with a little breast milk or water, or use a lanolin nipple cream, so that it adheres more easily to the breast. Pull back the edge of the nipple shield and centre the shield over the nipple. The baby's nose should be at the tulip shaped cut-out, so that it is touching the mums' skin.

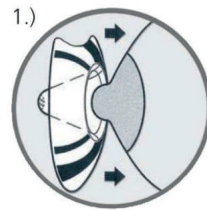

2. Release the edges of the nipple shield onto the breast.

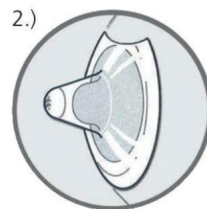

How to clean

Clean before first use and then once a day

Rinse with cold water (protein elimination)

Put a drop of washing-up liquid on the nipple shield, clean then wash with warm water (fat elimination)

Rinse thoroughly with clean water

Wrap in a clean towel

Store in a clean container until next use



In clinics:

Sterilisation: 121°C = 20 Min. / 134°C = 5 Min.

Thermic disinfection: 93°C = 10 Min.