# ardo amaryll manual breastpump

Instructions for use | Gebrauchsanweisung | Mode d'emploi |  
Istruzioni per l'uso | Instrucciones de uso | Indicações de  
utilização | Gebruiksaanwijzing | Bruksanvisning | Käyttöohje |  
Bruksanvisning | Brugsanvisning | Инструкция по эксплуатации |  
Instrukcja obsługi | Οδηγίες χρήσης | Kullanım Kılavuzu |  
دليل الاستعمال | 사용 설명서 | 使用说明

**Languages****Page**

<b>en</b>	Instructions for use	Amaryll manual breastpump	4–6
<b>de</b>	Gebrauchsanweisung	Amaryll Handmilchpumpe	7–9
<b>fr</b>	Mode d'emploi	Tire-lait manuel Amaryll	10–12
<b>it</b>	Istruzioni per l'uso	Tiralatte manuale Amaryll	13–15
<b>es</b>	Instrucciones de uso	Sacaleches manual Amaryll	16–18
<b>pt</b>	Indicações de utilização	Bomba de extração de leite manual Amaryll	19–21
<b>nl</b>	Gebruiksaanwijzing	Amaryll handkolf	22–24
<b>sv</b>	Bruksanvisning	Amaryll handbröstpump	25–27
<b>fi</b>	Käyttöohje	Käsikäyttöinen Amaryll-rintapumppu	28–30
<b>no</b>	Bruksanvisning	Amaryll håndpumpe	31–33
<b>da</b>	Brugsanvisning	Amaryll håndbrystpumpe	34–36
<b>ru</b>	Инструкция по эксплуатации	Ручной молокоотсос Amaryll	37–39
<b>pl</b>	Instrukcja obsługi	Laktator ręczny Amaryll	40–42
<b>el</b>	Οδηγίες χρήσης	Χειροκίνητο θήλαστρο Amaryll	43–45
<b>tr</b>	Kullanım Kılavuzu	Amaryll Manuel Süt Pompası	46–48
<b>ar</b>	دليل الاستعمال	Amayll مضخة اللبن اليدوية	49–51
<b>ko</b>	사용 설명서	애머리II 수동유축기	52–54
<b>zh</b>	使用说明	爱瑞手动吸乳器	55–57

## 1. Package contents

### Amaryll manual breastpump:

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**Amaryll Kombikit:** from a to d

## 2. Indications

The Amaryll manual breastpump is used for expressing, collecting and storing breast milk.

## 3. Instructions for use


### 3.1 Assembly

Important: Boil before first use  
(see Cleaning / sterilising).

1. Assemble according to the illustration
2. Place the membrane pot securely over the groove of the membrane con rod. Check that the smooth side of the membrane pot faces outwards.
3. Put on the casing cover in the desired position:
  - 3.1 Standard position
  - 3.2 Lever position left side
  - 3.3 Lever position right side

The mark on the casing cover must lie on top of the mark on the breast shell. Then rotate the casing cover clockwise.

4. The lever is mounted on the membrane con rod and by means of light pressure downwards clicks into the casing cover.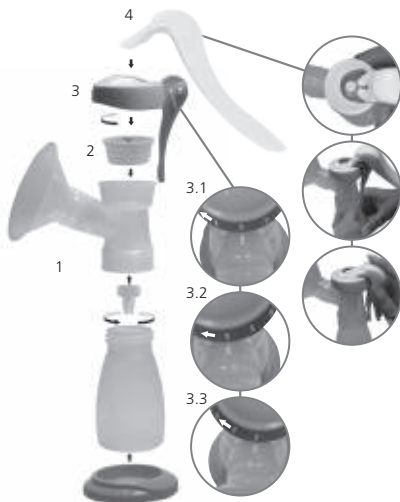

## 3.2 Expressing milk

- Wash your hands, sit down comfortably and relax.
- Place the casing cover in the desired position and tighten it.
- Centre the nipple so that when pumping it moves freely in the breast shell/breast shell insert and pump by activating the lever.
- Adjusting the suction intensity (vacuum): The further you press the lever, the greater the suction intensity. **1**
- Adjusting the speed interval: The faster you move the lever up and down, the higher the pumping frequency. **2**
- Our recommendation: Begin at a high speed interval and at low suction intensity in order to stimulate the milk ejection reflex. Once the milk is flowing, reduce the speed interval and increase the suction intensity





## 3.3 Choice of breast shell and Optiflow massage insert

If your nipple rubs uncomfortably on the inner side of the funnel, use a larger breast shell/breast shell insert. If the areola is also sucked into the funnel, use a smaller breast shell. The Optiflow massage insert can only be used with the 31 mm breast shell.

28 mm breast shell insert

Optiflow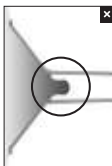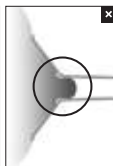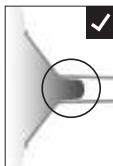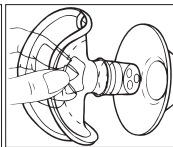

## 4. Storage of breast milk

Recommended storage time at room temperature: 3–4 hours at 16–29°C (60–85°F); in the refrigerator: 3 days at ≤4°C (39°F); in freezing compartment with door: 6 months to 1 year at <-17°C (0°F)

## 5. Cleaning / sterilising

At home: After each use, wash with washing-up liquid and rinse well. Once a day: disassemble and boil the components for 10 minutes, or put in the dishwasher, microwave (in the microwave bag) or steam steriliser, then dry and cover with a clean cloth. Do not clean the valve with the brush. The lever, casing cover and bottle stand can, but need not, be boiled.

In hospitals: Disassemble and sterilise at 121°C (250°F) for 20 minutes / 134°C (273°F) for 5 minutes. The lever, casing cover and bottle stand can, but need not, be sterilised. Can be disinfected by wiping.

## 6. Possible problems and troubleshooting

### Weak or no vacuum:

- Ensure that the Amaryll manual breastpump has been correctly assembled and is not damaged.
- Check whether the lip valve is cracked or stuck together.
- Make sure that the casing cover seals well.

### Pain when pumping:

- Ensure that the nipple is in the middle of the breast shell.
- Check that the breast shell size fits your nipples.
- If problems continue seek professional help.

## 7. Contraindications

None known.

**Safety regulations:** Only use after cleaning or sterilisation. After cleaning check for any damage.

## 8. General information

	Materials: Polypropylene (PP) / silicone		Relative humidity: 20–95%
	Temperature: Operation -10°C (14°F) to +40°C (104°F) / Transport -20°C (-4°F) to +50°C (122°F)		Air pressure: 900–1060 hPa
	Disposal: Can be safely disposed of in compliance with local regulations.		Keep packaging away from heat
	Packaging: Caution, fragile		Keep packaging away from moisture

Preferably store Optiflow in a dark place (silicone).

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Ardo medical AG