
en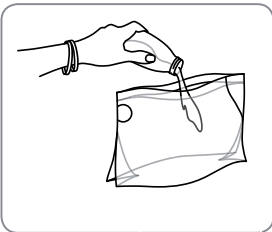



## INSTRUCTIONS FOR USE EASY CLEAN MICROWAVE BAG

### Instructions for use



- Clean all parts after use with warm soapy water and rinse them with clean water.
- Add 60 ml (2 ounces) of water to the bag.
- Place the parts to be steam-cleaned in the bag. Seal the bag tightly with the pressure fastening.
- Place the bag in the centre of the microwave.
- Set the microwave to the correct power and time and start it:
  - 1.5 minutes at > 1100 W
  - 3 minutes at 800 – 1100 W
  - 5 minutes at 600 – 750 W
- Open the microwave when it has finished and let the bag cool down for at least 2 minutes.
- Pick up the bag using the safe grip and remove it from the microwave.
- Hold the bag away from you and carefully pour out the remaining water through the steam vent.
- Open the bag and remove the steam-cleaned parts.
- The bag can be used up to 20 times. Tick a box after each use so you can check how many times are left.

### Warnings

- Always add at least 60 ml (2 ounces) of water to the bag to avoid damaging the parts.
- Do not overfill the bag.
- Do not place the bag into the microwave on a metal rack.
- Do not exceed the stated times for steam cleaning to prevent the parts from damage.
- If the microwave does not have a turntable, turn the bag halfway through.
- Caution: when removing the bag from the microwave and opening it, hot steam or water may escape.**
- Only use the bag with items deemed suitable for microwaving.
- Do not use the bag in a conventional oven and do not grill it.
- Store the bag out of the reach of children.
- To prevent limescale, you can add a drop of citric acid.
- Certain parts may discolour when used. This has no effect on their function.
- Follow the instructions for the microwave.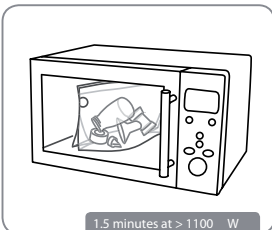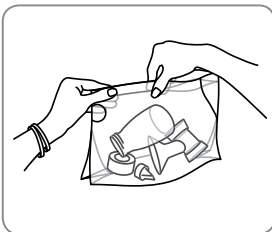

1.5 minutes at > 1100 W  
3 minutes at 800 – 1100 W  
5 minutes at 600 – 750 W

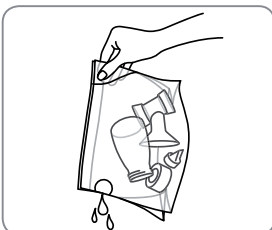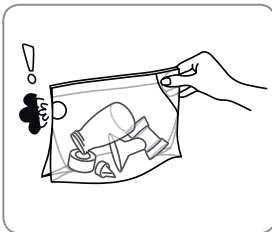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