ardo carum

electric breastpump

Instructions for use | Gebrauchsanweisung | Mode d'emploi
Istruzioni per l'uso | Instrucciones de uso | Instruções de utilização
Gebruiksaanwijzing | Bruksanvisning | Käyttöohjeet | Instrukcja
użycia | Οδηγίες χρήσης | Kullanım kılavuzu | تعليمات الاستخدام | 使用说明

	Languages		Page
en	Instructions for use	Carum – Breastpump for hospitals and hire	3-17
de	Gebrauchsanweisung	Carum – Milchpumpe für Kliniken und Vermietung	18-32
fr	Mode d'emploi	Carum – Tire-lait pour les maternités et la location	33-47
it	Istruzioni per l'uso	Carum – Tiralatte professionale per uso ospedaliero o per noleggio	48-62
es	Instrucciones de uso	Carum – Sacaleches para hospitales y alquiler	63-77
pt	Instruções de utilização	Carum – Bomba de extração de leite para uso hospitalar e aluguer	78-92
nl	Gebruiksaanwijzing	Carum – Borstkolf voor ziekenhuizen en verhuur	93-107
sv	Bruksanvisning	Carum – Bröstpump för sjukhus och uthyrning	108-122
fi	Käyttöohjeet	Carum – Rintapumppu sairaala- ja vuokrauskäyttöön	123-137
pl	Instrukcja użycia	Carum – Laktator do szpitali i wypożyczenia	138-152
el	Οδηγίες χρήσης	Carum – Θήλαστρο για νοσοκομεία και για ενοικίαση	153-167
tr	Kullanım kılavuzu	Carum – Klinikler ve Kiralama için Süt Pompası	168-182
ar	تعليمات الاستخدام	للمستشفيات وللتأجير CARUM مضخة الحليب	183-197
zh	使用说明	Carum – 医用或租赁用电动吸乳器	198-212

CARUM BREASTPUMP FOR HOSPITALS AND HIRE

Thank you for choosing the Carum breastpump from Ardo.

These instructions for use contain important and useful information about using the Carum.

When using electrical devices, all safety regulations must be followed. Please take your time. Read the instructions for use through carefully before using the Carum breastpump for the first time.



TABLE OF CONTENTS

1. Product location in the container	4	6. Useful tips	13
2. Intended Use	5	6.1. Expressing must never be painful.	13
2.1. Indications	5	6.2. Stimulating the milk let-down reflex	13
2.2. Contraindications	5	6.3. Efficient expressing	13
3. Safety Instructions	5	6.4. Increasing the volume of milk	13
4. Product Description	7	7. Accessories / replacement parts	14
5. Start-up / Operation	8	8. Error messages / status messages	14
5.1. Key	8	8.1. Other possible faults during pumping	14
5.2. Start-up	9	9. Cleaning / inspection / maintenance	15
5.2.1. Electrical connection	9	9.1. Cleaning / disinfection	15
5.2.2. Connecting the pumpset (applied part)	9	9.2. Inspection before switching on	15
5.2.3. Attaching the bottle holder	9	9.3. Maintenance	15
5.2.4. Attaching the Easy Freeze holder	9	9.3.1. Inspection checklist for hiring	15
5.3. Operation	10	9.3.2. Inspection checklist for use in hospitals	15
5.3.1. Switching the pump on and off	10	10. Warranty	16
5.3.2. Stimulation mode – Orange background	10	11. Technical information	16
5.3.3. Expression mode – Green background	11	11.1. Technical data	16
5.3.4. "Sensitive Programme"	11	11.2. Conformity	17
5.3.5. Triggering the let-down reflex again	11	11.3. EMC test record	17
5.3.6. Time measurement / Stopwatch	12	12. Transport and Storage	17
5.3.7. Dimmer function	12	13. Service life and disposal	17
5.3.8. Carum Battery	12		

The Carum is a vacuum pump that uses a double piston system. Using the control elements, every mother can adjust the settings for the vacuum and cycle to her own needs. The vacuum does not change when you adjust the cycle settings. The cycle also does not change when you adjust the vacuum settings. You must be able to read and understand these instructions for use.

1. Product location in the container

- a Carum breastpump
- b Connector openings for the pumpset
- c Control panel with LCD screen
- d Connector for power cord
- e Bottle holder
- f Easy Freeze holder (holds freezer bag)
- g Power cord
- h Carum container
- i Address field
- j Pumpset (not included in the delivery contents)
- k Instructions for use

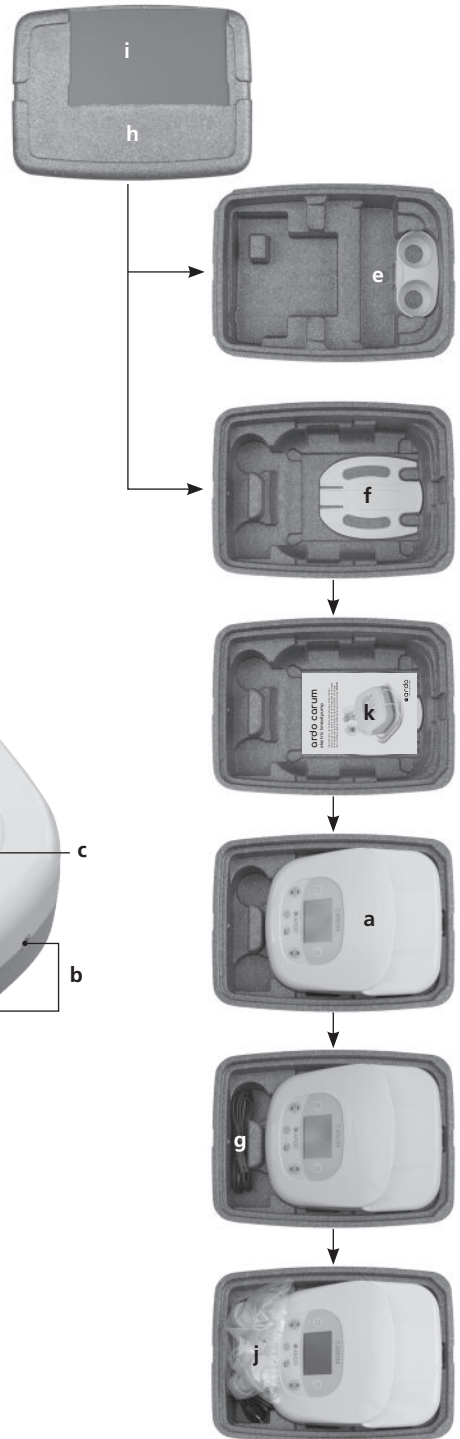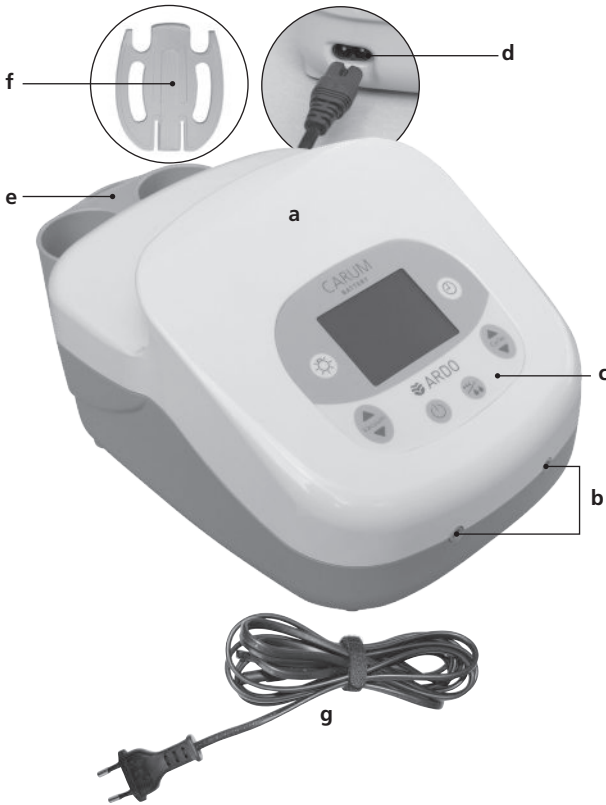2. Intended Use

2.1. Indications

The Carum breastpump is intended for use by breastfeeding women to express and collect their milk. It can be used as a single pump or as a double pump. The device is only intended for use indoors and is meant for several users.

2.2. Contraindications

The Carum must not be used during pregnancy because expressing milk can trigger contractions.

3. Safety Instructions

Important safety information

Please read the instructions for use carefully before using the pump for the first time. Keep these instructions for use in a safe place for later reference.

The Carum is an electric breastpump for use in hospitals and for hire. For your health and safety and to prevent damage to the device, the following safety rules must be followed.

Safety rules

DANGER: To avoid a fatal electric shock

- Only use the original power cord supplied by Ardo.
- Never use a faulty power cord.
- Never position an electrical device where it may fall into liquid.
- Never touch an electrical device that has fallen into liquid or is wet. Disconnect the device immediately from the power supply.
- Never use a device that has fallen into liquid or is wet but send the device to a service point or the manufacturer for inspection.
- Never use the Carum when taking a bath or showering.
- Never open the Carum! There is the risk of an electric shock.

WARNING: To avoid damage, electric shock, fire or injury

- The Carum is not intended for use outdoors.
- Never use the Carum if it is not functioning correctly, has been dropped, or is faulty.
- When using an electrical device, pay particular attention to children or people of impaired intelligence who are nearby; electrical devices are not toys.
- You must always connect the power cord to the Carum first and then plug it into the power socket.
- Never leave the Carum unattended when it is connected to mains power supply.
- Disconnect the Carum from the mains power supply after use.
- The device is only disconnected when the power plug is removed from the power socket.
- Position the Carum so that it can be easily disconnected from the mains power supply.
- Keep the Carum as well as the power cord and all accessory parts away from hot surfaces.
- Only use the Carum for the purposes listed in the section "Intended use".
- Only use original Ardo pumpsets and original Ardo parts.
- Do not insert any objects into openings or tubes on the Carum and do not allow any objects to fall into openings or tubes.
- Do not use the Carum in an environment in which oxygen is administered or sprays are used.
- The Carum must not be modified without the permission of the manufacturer.

WARNING: To avoid damaging your health or injuries

- Check the Carum and the associated parts before every use.
- Never use the Carum when you are sleeping or feeling tired.
- If expressing milk is uncomfortable or painful, stop pumping and ask for advice from the responsible qualified personnel.
- Do not use the Carum if you are pregnant because expressing milk can trigger contractions.
- Follow the instructions in the relevant section "Cleaning / Inspection / Maintenance" for cleaning.

Car adapter

- The Carum and the Carum Battery are not intended for use outdoors even with the car adapter.
- Only use the original electrically protected car battery cable supplied by Ardo.
- Ensure that your car has a 12 V DC connection.
- Do not express milk while driving, even as a passenger. This may lead to an accident.
- Disconnect the Carum from the electrical system after use or after recharging.

4. Product Description

- The Carum electric breastpump is intended for use in hospitals and for hire.
- The Carum can be used for single pumping and double pumping (expressing milk from both breasts at the same time).
- The cycles and the vacuum can be adjusted as required and independently of each other.
- The color screen provides visual orientation and the integrated stopwatch indicates the duration of pumping.
- To mimic the natural suckling behavior of the baby, milk can be expressed in two phases using the Carum: the stimulation mode and the actual expression mode. The stimulation mode starts the milk flow and the subsequent expression mode enables efficient pumping of the milk.
- The Carum starts with the stimulation mode and changes independently to the expression mode after two minutes. If required, you can switch to the expression mode early by pressing the mode button during this two minute period.
- To stimulate the milk flow again while expressing, you can easily change back to the stimulation mode at any time by pressing the mode button.
- The "Sensitive Programme" can be used if your nipples are sore to gently express milk with the minimum starting vacuum and minimum cycle.
- Ardo pumpsets create a 100% hygienic barrier between the breast milk and the breastpump. This closed system ensures that breast milk cannot end up in the tube of the pumpset or in the breastpump itself.
- The Carum Battery contains a rechargeable battery unit.
- The Carum and the Carum Battery can be operated with mains power or with an optional car cable in the car.

Carum Battery – the Carum with integrated rechargeable battery unit

The Carum Battery enables milk to be expressed anywhere and allows you to express milk beside the crib of an ill or newborn baby. Being close to your baby helps with the let-down reflex.

- The running time with a fully charged battery unit is about four hours (at max. vacuum and max. cycle). Recharging a flat battery unit takes about two hours.
- If the running time with a fully charged battery unit drops to less than one hour, the battery unit needs to be replaced at the closest service point or the manufacturer to restore optimal operation.
- If the device is connected to the mains power, the battery unit is recharged while milk is being expressed. The recharging time is prolonged in this case.
- If the Carum is connected to the electrical system of a vehicle via the car cable, the battery unit is not recharged while milk is being expressed. After switching the device off, the battery unit is charged via the electrical system of the car.

5. Start-up / Operation


5.1. Key

-  Follow the instructions for use
-  On / off button (stand-by button)
-  Vacuum button (to change the vacuum)
-  Cycle button (to change the cycle)
-  Mode button (to switch between stimulation / expression / "Sensitive Programme")
-  Dimmer button
-  Button for time measurement
-  Expression mode
-  Stimulation mode
-  Device with applied part BF
-  Conformity according to MDD 93 / 42 / EEC
0123
-  Manufacturer
-  The device is subject to the requirements of the WEEE Directive
-  Power supply unit
-  Alternating current
-  Device is protection class II
- IP21** Drip proof
- Li-Ion** Contains a lithium ion rechargeable battery unit
-  Device can be recycled

5.2. Start-up

5.2.1. Electrical connection




- Insert the power cord (a) into the Carum power connection (b) and connect to the power socket.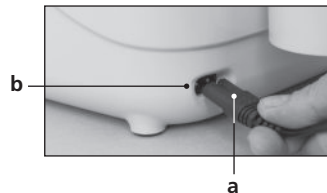

5.2.2. Connecting the pumpset (applied part)


The use of the pumpset is described in the separate operating instructions for the "Ardo Pumpset".

- Pumping with single pumpset:
Insert the tube connector (c) into one of the two connection openings on the pump.
- Pumping with double pumpset:
Insert both tube connectors (d) into the connection openings of the pump.
- The second opening (e) of the tube connector must always be closed for both single and double pumping.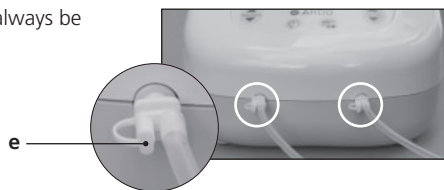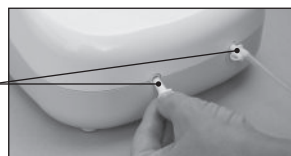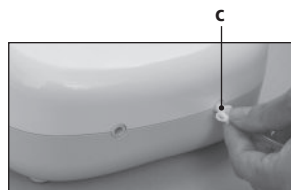


5.2.3. Attaching the bottle holder

The bottle holder (f) can be hung onto the back of the Carum or freely positioned.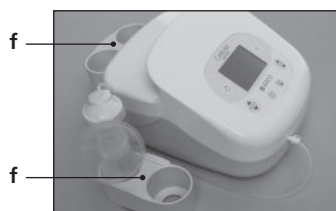

5.2.4. Attaching the Easy Freeze holder


The Easy Freeze holder (g) is used to hang a freezer bag or breast milk bottle on the pumpset. It can be hung onto the back in place of a bottle holder.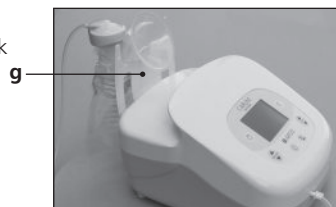

5.3. Operation


Vacuum and cycles can be adjusted independently of one another. Select the setting that is most comfortable for you.

- a** On / off button (stand-by button)
- b** Vacuum button (to change the vacuum)
- c** Vacuum indicator
- d** Dimmer button
- e** Dimmer information
- f** Time information
- g** Stopwatch information
- h** Button for time measurement
- i** Cycle indicator
- j** Cycle button (to change the cycle)
- k** Mode button (to switch between stimulation / expression / "Sensitive Programme")
- l** Expression mode
- m** Stimulation mode
- n** Information ribbon (n = e + f + g)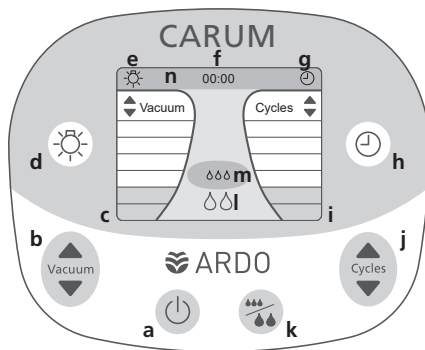

5.3.1. Switching the pump on and off

The Carum is switched on by pressing the on / off button once. The display starts with the Welcome page. To switch the Carum off, press the on / off button again.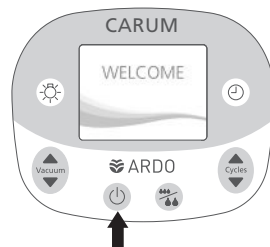


5.3.2. Stimulation mode – Orange background

When it is switched on, the Carum starts automatically in the stimulation mode (orange background) with a rapid cycle and a slight vacuum. This mode stimulates the let-down reflex. The automatic setting imitates the natural behavior of the baby. At the start of breastfeeding, babies suckle with rapid sucking movements and low suction strength which triggers the flow of breast milk.

The stimulation mode lasts for two minutes and then switches automatically to the expression mode.


The vacuum and the cycles can be increased or reduced by pressing the buttons repeatedly or holding them down. The selected settings are shown using bars. Adjust the vacuum and / or cycle so that you feel comfortable.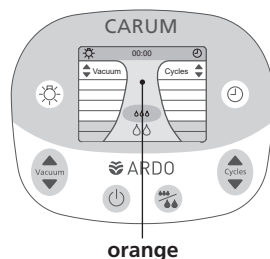

5.3.3. Expression mode – Green background

In the expression mode (green background) the Carum switches gently to a higher vacuum and a lower cycle. This automatic change imitates the natural behavior of the baby because it changes its suckling behavior as the breast milk starts to flow. The baby now sucks slower but with a greater suction strength.

The vacuum and the cycles can be increased or reduced by pressing the buttons repeatedly or holding them down. The selected settings are shown using bars. Again, you can adjust the vacuum and / or cycle in this mode so that you feel comfortable.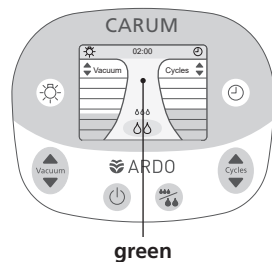

green


5.3.4. "Sensitive Programme"

The "Sensitive Programme" (green background) pumps with a barely detectable vacuum and cycle to enable gentle expressing of milk if you have sore nipples.

The vacuum and the cycle can be increased or reduced by pressing the buttons repeatedly or holding them down. The settings are shown using bars.

Setting the "Sensitive Programme":


1. Press the on / off button
2. Press the mode button while the Welcome page is being displayed.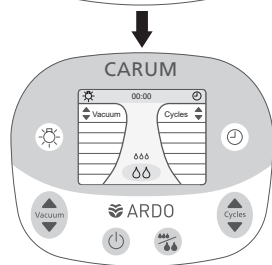

5.3.5. Triggering the let-down reflex again

In the expression mode (green background), you can switch to the stimulation mode (orange background) at any time by pressing the mode button again. This stimulates the milk flow again.

The mode button can also be used to change from the two minute stimulation mode to the expression mode ahead of time.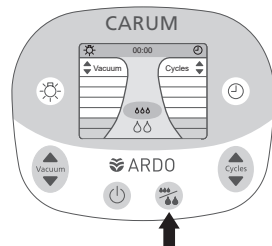


5.3.6. Time measurement / Stopwatch

The time measurement starts with switching on the Carum and is displayed in the information ribbon.


- The time measurement is stopped by pressing the Time button once.
- By pressing the Time button again, the clock is reset to 00:00.
- Pressing the Time button again starts the time measurement once more.

The time measurement is retained when changing modes but is reset to 00:00 when the Carum is switched off.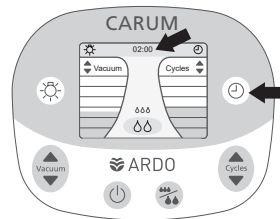

5.3.7. Dimmer function


The brightness of the display can be reduced by pressing the Dimmer button once; at the same time the light bulb symbol in the information ribbon fades out. By pressing the button again the display returns to its normal brightness.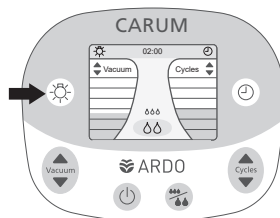

5.3.8. Carum Battery

The Carum Battery with its integrated battery unit is identified on the front panel by "BATTERY".

The battery unit charge status is displayed as a battery symbol in the information ribbon. During recharging, the bars move from left to right until the battery unit is fully recharged.

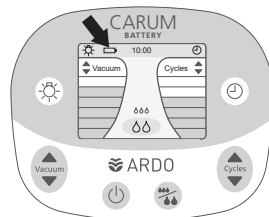6. Useful tips

6.1. Expressing must never be painful.

- Check that the size of the breast shell fits the size of your nipple (see instructions for use for the pumpset).
- Ensure that your nipple is centered in the breast shell.
- If your milk is not flowing, a breast massage while expressing may be helpful.
- If your nipples are sore or your breasts are painful, preferably use the "Sensitive Programme" without stimulation beforehand. The "Sensitive Programme" starts pumping milk using a very gentle and low vacuum.

6.2. Stimulating the milk let-down reflex

Because the let-down reflex when expressing can occur later than with natural breastfeeding, the following tips may be helpful:

- Gently massage the breast before expressing milk.
- Drink a warm drink.
- Think about your baby and relax.
- Look at photos of your baby.
- Before expressing milk, place a warm "Ardo Temperature Pack" or a warm, damp cloth on the breast.
- Warm the breast shell, under running warm water for example, before using it.

6.3. Efficient expressing

- By massaging the breast while expressing, you can obtain more milk and improve the milk flow.
- Select a vacuum that makes you feel comfortable.

6.4. Increasing the volume of milk

- Pump both breasts at the same time (double pumping).
- Double pump for the first ten minutes or until milk no longer flows. Then switch to single pumping and pump each breast for five minutes. With the hand that is now free, massage the breast gently before pumping.

7. Accessories / replacement parts

Item	Item number
Euro power cord	63.00.130
UK power cord	63.00.131
US power cord	63.00.132
Mobile stand, optional	63.00.129
Bottle holder	63.00.127
Easy Freeze holder	63.00.128
Various pumpsets and other accessories as well as replacement parts	www.ardomedical.com

8. Error messages / status messages

 Error	<p>A fault in the Carum or the Carum Battery is indicated on the display by the adjacent symbol. Please switch the device off and switch it on again. If the fault persists, please have the device inspected by a service point or the manufacturer.</p>
 	<p>A flat battery unit in the Carum Battery is indicated on the screen by the adjacent symbol. When this symbol appears on the display, the Carum Battery switches off automatically after about 5 seconds. Insert the power cord into the device and connect it to the mains power to recharge the battery unit.</p>

8.1. Other possible faults during pumping

No vacuum or vacuum is too weak

Check that

- you are using an original Ardo pumpset. (Always express milk using an Ardo pumpset.)
- the second opening on the tube connector is closed.
- the tube connector is firmly seated in the connection opening of the pump.
- the pumpset is correctly assembled and is not damaged. (See instructions for use for the pumpset)

Carum does not function / display remains dark

Check that

- the power cord and its socket are visually in order and are not faulty. Never use a faulty cord or a cord with faulty plugs!
- the power cord is correctly plugged into the Carum and into the power socket.
- the Carum works on another power socket.
- another device works on the same power socket.

If you confirm that the Carum is faulty, please have the device inspected by a service point or the manufacturer.

9. Cleaning / inspection / maintenance

If the cleaning and inspections are not carried out as required, there is a risk of infection or of the device developing a fault.

9.1. Cleaning / disinfection

- Always disconnect the Carum from the mains power supply before cleaning!
- The Carum is not waterproof which is why it must never be immersed in liquids! Never clean the Carum in a dishwasher, hand basin, or under running water.
- You must also ensure that no cleaning liquid enters the connection openings for the pumpset or the electrical connections.

Cleaning at home

- As required, wipe the outside of the Carum with a damp cloth. You may use a little standard dishwashing detergent if necessary but do not use any abrasive cleaners.
- Clean the pumpset in accordance with the directions in the relevant instructions for use.

Cleaning in the hospital

- Carum: Wipe disinfection of the surfaces with a non-phenolic disinfectant. Do not use abrasive cleaning agents.
- Mobile stand for the Carum: Wipe disinfection of the surfaces with a non-phenolic disinfectant. Do not use abrasive cleaning agents.
- Clean the pumpset in accordance with the directions in the relevant instructions for use.

9.2. Inspection before switching on

- The housing and the display are intact.
- The power cord and the power socket are not damaged.
- The pumpset is clean, not faulty, and correctly assembled.
- The pumpset is correctly connected to the connection opening of the Carum.
- Any accessories are clean, not faulty, and correctly assembled.
- Mobile stand (optional): Check that the screws are tightened.

9.3. Maintenance

The Carum does not in principle need any maintenance. The following inspections are, however, recommended to ensure perfect functioning:

9.3.1. Inspection checklist for hiring

Visually inspect the pump for cleanliness	After every hire
Visually inspect the pump for completeness	After every hire
Visually inspect the pump for damage	After every hire
Simple functional check of the pump (if the pump runs, the battery unit is charged)	After every hire
Vacuum check carried out by trained service personnel	At least every 2 years

9.3.2. Inspection checklist for use in hospitals

Visually inspect the pump for cleanliness and damage	Before and after each use
Vacuum check carried out by trained service personnel	At least every 2 years

10. Warranty

The warranty period of the Carum breastpump is 3 years from the date of purchase; the dated receipt is considered the certificate of warranty.

The warranty period for the rechargeable lithium ion battery unit for the Carum Battery is 1 year from the date of purchase; the dated receipt is considered the certificate of warranty.

General warranty conditions

Ardo medical AG guarantees that the devices it manufactures are free of material and manufacturing defects. With proper use, material that develops a defect during the warranty period is replaced free of charge. To protect the right to make a warranty claim and to ensure that the device functions without fault, the instructions for use must be followed and only accessories from Ardo medical AG may be used. The warranty benefit is voided if the device has been modified in any way.

11. Technical information

11.1. Technical data	
Dimensions of the pump LxWxH	302 × 240 × 180 mm
Pump weight	3.0 kg
Pump weight with rechargeable battery unit	3.7 kg
Power cord length	3 m
Mains voltage / frequency	100–240 VAC ± 10% / 50/60 Hz
Power consumption	135 VA
Mains fuse input P / N	T 2.5 A / 250 VAC
Connector for car cable	12 V DC
Operating conditions	5–40°C (41–104°F) 10–93% RH, not condensing 700–1060 hPa
Degree of protection (pumpset is an applied part)	BF
Protection class	Class II
Classification as per MDD 93/42/EEC	Class IIa
Dimensions of the mobile stand LxWxH	425 × 425 × 720 mm
Ground clearance	105 mm
Wheel diameter	75 mm
Mobile stand weight	7.0 kg

	Vacuum ($\pm 5\%$ ¹⁾)	Cycle (± 3)
Stimulation mode		
Start value	60 mbar 6 kPa 45 mmHg	90 / min
Setting values	30–150 mbar 3–15 kPa 22–113 mmHg	72–120 / min
Expression mode		
Start value	*	45 / min
Setting values	30–330 mbar 3–33 kPa 22–248 mmHg	30–60 / min
"Sensitive Programme"		
Start value	30 mbar 3 kPa 22 mmHg	30 / min
Setting values	30–330 mbar 3–33 kPa 22–248 mmHg	30–60 / min

* The expression mode uses the setting value last selected by the user in the stimulation mode and increases the vacuum by 30 mbar / 3 kPa / 22 mmHg

¹⁾ The vacuum values are independent of the actual atmospheric pressure.

11.2. Conformity	11.3. EMC test record
<p>As per the Medical Device Directive 93/42 EEC, EN 60601-1, EN 60601-1-2, EN ISO 10079-1, EN 60601-1-6, EN 60601-1-11, 2011/65/EU (RoHS)</p>  <p>CLASSIFIED C UL US E358823</p> <p>MEDICAL – GENERAL MEDICAL EQUIPMENT AS TO ELECTRICAL SHOCK, FIRE AND MECHANICAL HAZARDS ONLY IN ACCORDANCE WITH ANSI/AAMI ES60601-1 (2005, C1:09, A2:10), CAN/CSA C22.2 No.60601.1 (2008, R:2011)</p>  <p>0123</p>	<p>See page 213</p>

12. Transport and Storage

The original packaging must be used to transport and store the Carum and the Carum Battery. We recommend storing the Carum protected against UV light (e.g., sunlight, halogen lamps), heat, and dust.

Carum Battery: Please obtain information from the particular airline about transport requirements when flying.

Temperature	-20°C to +70°C (-4°F to 158°F)
Relative air humidity	10–93% RH, not condensing
Air pressure	700–1060 hPa

13. Service life and disposal

The service life of the device with proper use is 10 years excluding wear parts. This device contains electronic components and a rechargeable lithium ion battery unit.

At the end of the service life of the device, the device and its accessories must be disposed of in accordance with the locally applicable regulations.

Used rechargeable battery units must not be disposed of in household trash but instead returned to the point of purchase or disposed of in accordance with locally applicable regulations.

The device is subject to the requirements of the WEEE Directive.

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

TEST REPORT**IEC / EN 60601-1-2****Part 1-2: Electromagnetic Compatibility Requirements and Tests**

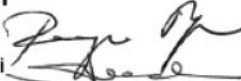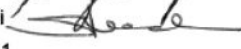Report reference No:	E1541-05-11	
Tested by:	B. Belegu	
Approved by management:	Dr. F. Stucki	
Date of issue:	26 April 2011	
Number of pages:	1 page	
Testing laboratory:	QUINEL AG	 S Schweizerischer Prüfstellendienst T Service Suisse d'essai S Servizio di prova in Svizzera S Swiss testing service STS 037
Address:	Grundstr. 2 CH-6343 Rotkreuz	
Testing location:	Rotkreuz Tel. 041-799 47 04	
Applicant's Name:	Ardo Medical AG, Mr. Christian Waser	
Address:	Gewerbstrasse 19, CH-6314 Unterägeri	
Manufacturer:	Ardo Medical AG	
Address:	Gewerbstrasse 19, CH-6314 Unterägeri	
Test specification:		
Standards:	IEC 60601-1-2:2007, EN 60601-1-2:2007	
Test procedure:	Type testing for Swiss and EU legal requirements	
Procedure deviation:	None	
Non-standard test method:	None	
Test-specification:	The used test setup fulfils the specification described in the relevant standards	
Test item description:	Breast Pump	
Trademark:	Ardo	
Model and/or type reference:	Carum	
Ratings:	100V-240V AC, 50Hz-60Hz	
Date of receipt of the test item(s):	06 April 2011	
Summary of testing:	Passed	
Applied standards:		
Title	Standard (up dated)	
RF disturbances	CISPR 22:2008, EN 55022:2010	
RF conducted disturbances	IEC 61000-4-6:2008, EN 61000-4-6:2009	
Radiated electromagnetic field	IEC61000-4-3:2006+A1:2007+A2:2010, EN 61000-4-3:2006+A1:2008+A2:2010	
Power frequency magnetic field	IEC 61000-4-8:2009, EN 61000-4-8:2010	
Electrical fast transient burst	IEC 61000-4-4:2004+A1:2010, EN 61000-4-4:2004+A1:2010	
Electrostatic discharge	IEC 61000-4-2:2008, EN 61000-4-2:2009	
Surge	IEC 61000-4-5:2005, EN 61000-4-5:2006	
Voltage dips - interruptions	IEC 61000-4-11:2004, EN 61000-4-11:2004	
Harmonic current emissions	IEC 61000-3-2:2005+A1:2008+A2:2009, EN 61000-3-2:2006+A1:2009+A2:2009	
Voltage fluctuations and flicker	IEC 61000-3-3:2008, EN 61000-3-3:2008	
Acceptance criteria/operating conditions:		
During and after the test the apparatus should show no damage or loss of information and should function according to specifications.		
This Test Report is, together with the Test Report E1541-05a-11 and E1541-05b-11, valid for the following types: Carum 1, Carum 2		

Summary of test for Electromagnetic Compatibility

The test of electromagnetic compatibility according to IEC / EN 60601-1-2 for this device has shown that in the intended use no endangering of the functional safety or a failure of this device due to electromagnetic influences are to be expected.

This device uses HF-energy exclusively for its internal function.

Its HF-transmission is therefore very small, and it is therefore unlikely that adjacent electronic equipment will be affected.

Since the propagation of electromagnetic variables is affected by absorptions and reflections from structures and / or by objects and people, the following must be observed:

This device should not be operated at a short distance from other devices. Do not place any other devices on the housing of this device. If it must operate at a short distance from other devices, the device should be observed to verify its intended performance. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the device.

Mobile radio equipment or mobile phones should not be used at a short distance from this device or its mains power cable.

Interference may occur in the vicinity of equipment marked with the symbol adjacent.

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