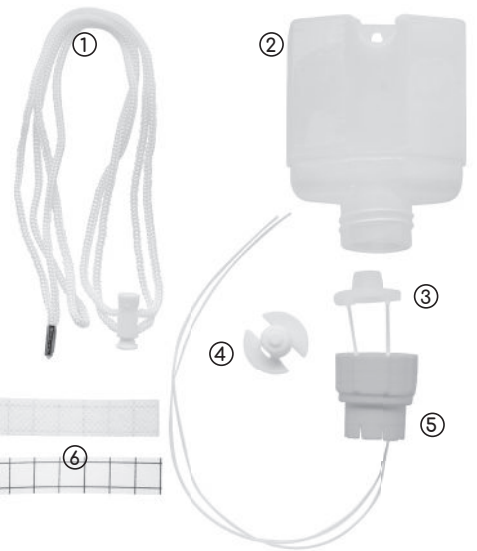
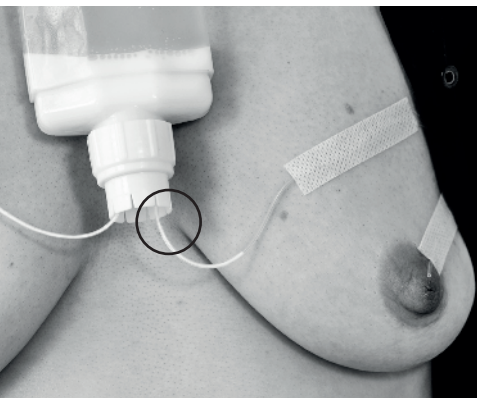
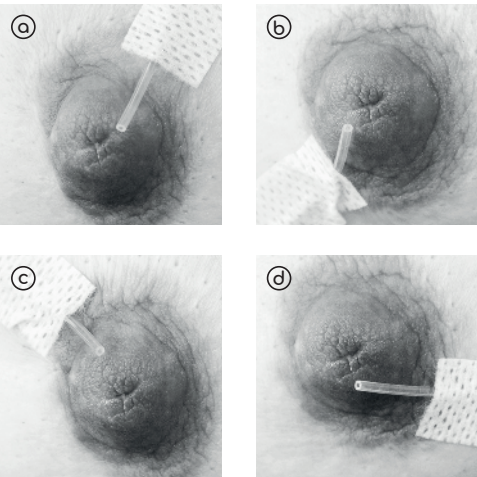
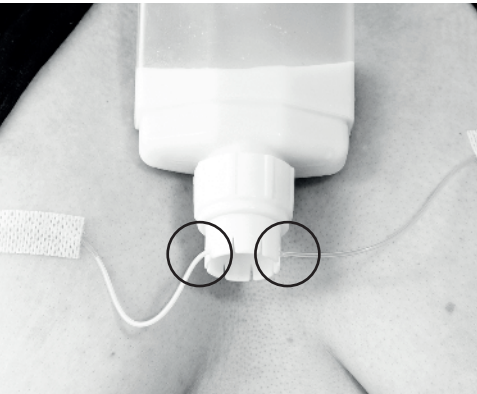
- ① 1× Umhängeband / Lanyard / Lanyard / Cordino / Cordón / Sleutelkoord
- ② 1× Milchflasche / Milk bottle / Bouteille de lait / Bottiglia di latte / Botella de leche / Melkfles
- ③ 3× Silikonventil mit Schläuchen / Silicone valve with tubes / Valve en silicone avec tubes / Valvola in silicone con tubi / Válvula de silicona con tubos / Siliconen ventiel met slangetjes
- ④ 1× Ventilkappe / Valve cap / Capuchon de valve / Tappo della valvola / Tapa de la válvula / Ventieldop
- ⑤ 1× Verbindungsring / Connection ring / Anneau de raccordement / Anello di connessione / Anillo de conexión / Verbindingsring
- ⑥ 50× Klebestreifen / Adhesive strips / Bande adhésive / Strisce adesive / Tiras adhesivas / Zelfklevende strips

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Intended use
The Ardo supplemental nursing system is a special breast-feeding aid, which can be used to combine additional feeding and breastfeeding of your baby. It is mainly used in situations in which:

- the baby requires nutrition in addition to breast milk (e.g. for premature babies),
- the mother is not (yet) producing enough milk,
- the baby has difficulties with sucking or
- adoptive or foster parents would like to develop a breastfeeding relationship.

Safety instructions

- Ideally, milk should be fed at body temperature (max. 37 °C); never heat it in the microwave!
- Check the system before each use and dispose of it if damaged or soiled.
- Keep the system out of the reach of children.
- For hygiene reasons, this product is only intended for use by one person.

Use

- Before use, wash your hands thoroughly using soap and water.
- Pass the metal rod of the lanyard (1) through the eyelet on the bottle (2) and pull the lanyard tight.
- Fit the silicone valve with both tubes (3) onto the valve holder (4) and press firmly.
- Now feed the tubes through the attachment ring (5) from above and push the valve with holder into it.
- Pour the desired amount of milk into the bottle (2) and screw the attachment ring with valve onto the bottle.
- Clamp both tubes in the grooves of the attachment ring to block the flow of milk, before turning the bottle over (Figure 1).
- Hang the bottle around your neck at breast height and position the tube so that it is level with the nipple or barely protrudes away from it Figure 2). Secure the position of the tube using the adhesive strips (6).
- If you wish to feed from just one breast, you can affix the second tube onto the bottle with the opening facing up. To breastfeed from both sides, attach the tubes at both breast.

Depending on the breastfeeding position, you should guide the tube in the appropriate direction (in the example of the right breast):

- Football hold**, tube against the upper lip (a), tube against the lower lip (b)
- Cradle hold**, tube against the upper lip (c), tube against the lower lip (d)

- When your baby begins to suck at the breast with the tube in the mouth, release the tube from the attachment ring to allow milk to flow (Figure 3). It is important that the baby is always actively sucking at the breast and not only on the tube.
- After 10–20 seconds, release the second tube from the attachment ring so that the underpressure in the bottle is released and air can flow into the bottle (Figure 4). Once air bubbles visibly rise in the bottle, this means your baby is now receiving milk from the bottle.

The flow speed of milk through the tube can be adjusted by changing the position of the bottle: the higher the bottle hangs away from the end of the tube, the faster the milk will flow. Alternatively, the milk flow can be altered with a light pressure on the bottle.

When using a supplemental nursing system, it is always advisable to seek professional advice from a lactation consultant or a medical professional.

Cleaning – before using for the first time and immediately after each use

- Disassemble the system into its individual parts and clean all parts with cold water.
- Fill the bottle with cold water, fully assemble the system and squeeze the bottle to flush water through the tube.
- Repeat these steps with warm soapy water and then with clean, cold water.
- Place all parts onto a clean cloth to allow them to air dry. Ensure the system is clean and dry before storing

Disinfection – before using for the first time and once daily

- Disinfect all parts in a pan of boiling water for 5 minutes or in the microwave (e.g. with the Ardo Easy Clean).
- Place all parts onto a clean cloth to allow them to air dry. Ensure the system is clean and dry before storing.

Disposal
In accordance with local regulations.

Complies with the requirements of Regulation (EC) No. 1935/2004 for food-safe products.

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